Prime&Bond® universal Universal Adhesive

Caution, consult
Instructions for Use




		
20 sec. Agitate slightly	≥ 5 sec. Remove solvent	Cure ≥ 550 mW/cm ² : 20 sec. ≥ 800 mW/cm ² : 10 sec.

THE DENTAL
SOLUTIONS
COMPANY™

Step-by-step instructions: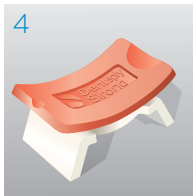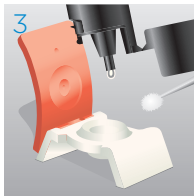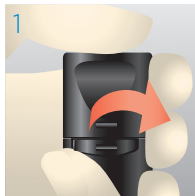

Preparation

Pull off originality seal.

Flip Top Bottle can be opened and closed with one hand. Do not screw off the cap.

Dispense **Prime&Bond® universal** into a **CliXdish™**.


In a closed **CliXdish™ Prime&Bond® universal** remains usable for up to 30 minutes.

To close the bottle, press down the cap until it locks in place with a perceptible “click”.


Cleaning &
Disinfection
Disassemble
CliXdish™
for reprocessing.

OPTIONAL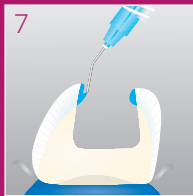

Etching

Start etching at
enamel margins.

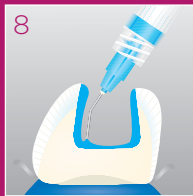Extend to dentin.


Rinsing

Remove acid gel
and rinse.


Remove excess
water.

Do not desiccate!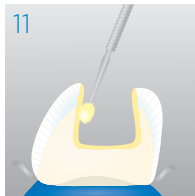


Application

Apply Prime&Bond® universal to all cavity surfaces.
Avoid pooling.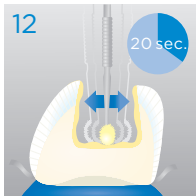

Keep Prime&Bond® universal slightly agitated for 20 seconds.

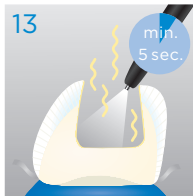Evaporation

Evaporate solvent with air for at least 5 seconds.

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Prime&Bond® universal

Universal Adhesive

CAUTION: For dental use by dental professionals only.

USA: Rx only.

Content	Page
1 Product description	2
2 Safety notes	3
3 Step-by-step instructions	5
3.1 Placement of direct light-cured restorations	5
3.2 Cementation of light-cured resin-cemented veneers	7
3.3 Repair of composite, ceramic and amalgam restorations with light-cured composites	7
3.4 Cavity varnish for use with fresh amalgam	7
3.5 Cementation of indirect restorations using Calibra® Ceram or Calibra® Universal	8
3.6 Placement of direct dual-cure/self-cure composite restorations and core build-ups	8
3.7 Cementation of indirect restorations with other dual-cure and self-cure resin cements	10
3.8 Endodontic post cementation	11
4 Hygiene	12
5 Lot number, expiration date and correspondence	13

1 Product description

Prime&Bond® universal universal adhesive is a combined Etch&Rinse (Total-Etch), Self-Etch and Selective-Etch adhesive. It offers a simple adhesive application technique for both direct and indirect indications and bonds to enamel, dentin, composites, zirconia and metals.

1.1 Indications

- Direct, light-cured composite and compomer restorations.
- Light-cured resin cemented veneers.
- Composite, ceramic and amalgam repairs.
- Cavity varnish for use with fresh amalgam.
- Indirect restorations and endodontic posts cemented with Calibra® Ceram or Calibra® Universal.

In combination with Dentsply Sirona Self Cure Activator:

- Direct dual-cure/self-cure composite restorations and core build-ups.
- Cementation of indirect restorations and endodontic posts using dual-cure/self-cure resin cements.

1.2 Contraindications

- Use in patients with a history of allergic reaction to (meth)acrylate resins or any of the components.
- Direct application to dental pulp (direct pulp capping).

1.3 Delivery forms

Prime&Bond® universal universal adhesive is available in:

- Tri-layer bottle with flip-top-cap

1.4 Composition

- Multifunctional acrylate
- Water
- Isopropanol
- Acidic acrylate
- Phosphoric acid-modified acrylate resin (PENTA)
- Bifunctional acrylate
- Camphorquinone (photoinitiator)
- Initiator
- 4-(Dimethylamino)benzonitrile (photoaccelerator)
- Stabilizer

1.5 Compatible materials

Prime&Bond® universal universal adhesive is compatible with conventional (meth)acrylate-based visible light-cured restorative and cement materials.

Prime&Bond® universal universal adhesive is compatible with Dentsply Sirona Calibra® Ceram, Calibra® Universal, and Calibra® Veneer (see respective Instructions for Use for detailed instructions).

When used with Dentsply Sirona Self Cure Activator Prime&Bond® universal universal adhesive is compatible with dual-cure/self-cure resin cements, composites, and core build-up materials.

2 Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

Safety alert symbol.

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

Information symbol.

This information symbol is used to highlight important information for the correct use.

2.1 Warnings

The material contains polymerizable (meth)acrylate monomers and 4-(Dimethylamino)benzonitrile, which are allergenic, and may be irritating to skin, eyes, and oral mucosa, as well as cause allergic contact dermatitis in susceptible persons. The material is acidic and may cause burns.

- **Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes rinse with plenty of water and seek medical attention.

- **Avoid skin contact** to prevent irritation and possible allergic response. In case of contact, red-dish rashes may be seen on the skin. If contact with skin occurs, remove material with cotton and alcohol and wash thoroughly with soap and water. In case of skin sensitization or rash, discontinue use and seek medical attention.
- **Avoid contact with oral soft tissues/mucosa** to prevent inflammation. If accidental contact occurs, remove material from the tissues. Flush mucosa with plenty of water and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.

2.2 Precautions

This product is intended to be used only as specifically outlined in these Instructions for Use. Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.

- If refrigerated, allow material to reach room temperature prior to use.
- Use protective measures for the dental team and patients, like glasses and rubber dam, in accordance with local best practice.
- The bottle cannot be reprocessed. To prevent the bottle from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the bottle is handled offside the dental unit with clean/disinfected gloves.
- Use in a well-ventilated area. Avoid inhaling vapor.
- Flammable: Keep away from sources of ignition. Take precautionary measures against static discharges.
- Prime&Bond® universal universal adhesive is a light-cured material. Protect from ambient light.
- Applicator Tips are intended for single use only. Discard after use. Do not reuse in other patients to prevent cross-contamination.
- Unprepared enamel must be etched using phosphoric acid before application of Prime&Bond® universal universal adhesive.
- Contact with saliva, blood and sulcus fluid during application may cause failure of the restoration. Use adequate isolation, like rubber dam.
- Tightly close bottle immediately after use.
- Interactions:
 - Do not use eugenol- or hydrogen peroxide-containing materials in conjunction with this product since they may interfere with hardening and cause softening of the product.
 - Avoid the product saturating the gingival retraction cord. If it soaks the cord, it may set hard and bond the cord to the underlying tooth surface making removal difficult.
 - If mineral-impregnated (e.g., ferric compounds) retraction cords and/or hemostatic solutions are used in conjunction with adhesive procedures, marginal seal may be adversely affected, allowing microleakage, subsurface staining and/or restoration failure. If gingival retraction is necessary, use of plain, non-impregnated cord is recommended.

2.3 Adverse reactions

- Eye contact: Irritation and possible corneal damage.
- Skin contact: Irritation or possible allergic response. Reddish rashes may be seen on the skin.
- Contact with Mucous Membranes: Inflammation (see [2.1 Warnings](#)).

2.4 Storage conditions

Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.

- Keep the product out of direct sunlight and store in a well-ventilated place.
- Store at temperatures from 2 °C to 24 °C (35 °F-75 °F). Use the product at room temperature.
- Do not use after expiration date.

3 Step-by-step instructions

3.1 Placement of direct light-cured restorations

3.1.1 Isolation, cleaning and conditioning of enamel and dentin

1. Use adequate isolation, like rubber dam.
2. Clean unprepared enamel and dentin with a rubber cup and pumice or a cleaning paste, like Nupro® Prophylaxis Paste.
3. Wash prepared and unprepared enamel and dentin thoroughly with water spray and blot- or lightly air-dry. Do not desiccate dentin.

3.1.2 Pulp protection

1. In deep cavities, cover the dentin close to the pulp with a hard-setting calcium hydroxide liner (Dycal® calcium hydroxide liner, see Instructions for Use) leaving the rest of the cavity surface free for bonding.

3.1.3 Acid conditioning of enamel and dentin

Prime&Bond® universal universal adhesive offers operators the choice of enamel and dentin pre-treatment:

➡ Self-Etch mode

1. No etching with phosphoric acid, proceed directly to application of the adhesive.

➡ Selective enamel-etch mode

1. Apply phosphoric acid (e.g., Conditioner 36) according to manufacturer's instructions.
2. Gently extrude conditioner gel to the enamel (margins) only.
3. For best results, condition **enamel for at least 15 seconds**. Any incidental contact of conditioner gel with dentin should be limited to 15 seconds or less.
4. Remove gel with aspirator tube and/or vigorous water spray and rinse conditioned areas thoroughly for at least 15 seconds.
5. Remove rinsing water completely by blowing gently with an air syringe or blot-dry.
6. Do not desiccate dentin.
7. Proceed immediately to application of the adhesive.

➡ Etch-and-rinse mode

1. Apply phosphoric acid (e.g., Conditioner 36) according to manufacturer's instructions.
2. Gently extrude conditioner gel to the cavity surfaces starting at the enamel margins.
3. For best results, condition **enamel for at least 15 seconds and dentin for 15 seconds or less**.
4. Remove gel with aspirator tube and/or vigorous water spray and rinse conditioned areas thoroughly for at least 15 seconds.
5. Remove rinsing water completely by blowing gently with an air syringe or blot-dry.
6. Do not desiccate dentin.
7. Proceed immediately to application of the adhesive.

Contamination.

Restoration failure.

1. Once the surfaces have been properly treated, they must be kept uncontaminated.
2. If salivary contamination occurs after acid conditioning, thoroughly clean with vigorous water spray, dry and apply adhesive as described in chapters [3.1.4 Dosage](#) to [3.1.6 Light-curing of adhesive](#) without repeating the acid conditioning step.

3.1.4 Dosage

Flip-top Bottle:

1. Pull off originality seal.
2. To open, take the bottle in the hand, place a thumb in the thumb recess and push the cap up until it remains in the open position.
3. Hold the bottle upside-down in a vertical position and dispense 1-2 drops of adhesive into a CliXdish™ Mixing Well or standard dappen dish or mixing well.
4. If necessary clean dropper of the bottle using a soft paper tissue.
5. Carefully close the bottle again by applying pressure to the lever from above. The cap snaps into place when pressed closed with a perceptible "click".

Technique Tip: CliXdish™.

Material in a closed CliXdish™ Mixing Well will remain usable for **up to 30 minutes**.
Material dispensed into a standard mixing well must be used immediately.

3.1.5 Application of adhesive to enamel and dentin

1. Use a new Applicator Tip to apply adhesive to completely wet the surfaces to be treated. If necessary, rewet Applicator Tip. Avoid pooling of the adhesive.
2. Keep the adhesive slightly agitated for 20 seconds.
3. Disperse adhesive and remove solvent with clean, dry air from an air-water syringe. Treat every surface with a **moderate air flow** for at least 5 seconds, until achieving a glossy and uniform layer. Avoid dry spots and splashing of adhesive by using a too strong air flow. Light-cure adhesive immediately afterwards, as described in [3.1.6 Light-curing of adhesive](#).

Insufficient evaporation of solvent.

Inadequate polymerization.

1. Follow strictly the instructions of the steps above.

3.1.6 Light-curing of adhesive

1. Light-cure adhesive with a suitable curing light¹. Cure adhesive **for 10 seconds** when using a curing light with a minimum irradiance of 800 mW/cm².
Cure adhesive **for at least 20 seconds** when minimum irradiance is between 550 mW/cm² and 800 mW/cm².

Insufficient curing.

Inadequate polymerization.

1. Check compatibility of curing light.
2. Check curing time.
3. Check minimum irradiance.

2. Proceed immediately to placement of the direct restoration or the preparation and cementation of indirect restoration as described in the following chapters.

3.1.7 Placement of direct restorations with light-cured composites and compomers

1. After application and light-curing of adhesive, place restorative following the Instructions for Use of the used composite or compomer material.

¹ Curing light designed to cure materials containing camphorquinone (CQ) initiator. Peak of spectrum in the range of 440-480 nm and irradiance of 800-2000 mW/cm².

2. Ensure that the cured adhesive surfaces remain uncontaminated before applying the direct restorative material. If salivary contamination occurs, thoroughly clean with vigorous water spray, dry and reapply adhesive as described in chapters [3.1.4 Dosage](#) to [3.1.6 Light-curing of adhesive](#) without repeating the acid conditioning step.

3.2 Cementation of light-cured resin-cemented veneers

Application of adhesive.

Application of adhesive to the inner surface of glass ceramic restorations treated with silane is not required.

1. Clean enamel and dentin, apply and light-cure adhesive as outlined in [3.1.1 Isolation, cleaning and conditioning of enamel and dentin](#) to [3.1.6 Light-curing of adhesive](#).
2. Treat bonding surface of restoration as follows:
 - **In case of glass ceramics** etch with hydrofluoric acid according to the manufacturer's instructions and apply silane (e.g., Calibra® Silane Coupling Agent) following manufacturer's instructions.
 - **In case of zirconium dioxide or composite**, sandblast with an intraoral microetcher (50 µm alumina). Rinse with water for 15-20 seconds, air dry. **Apply** Prime&Bond® universal universal adhesive to bonding surfaces of restorations to enhance adhesion. **Evaporate** as outlined in [3.1.5 Application of adhesive to enamel and dentin](#).
3. Proceed immediately to cementation of indirect restoration following the Instructions for Use for light-cured esthetic resin cement.

3.3 Repair of composite, ceramic and amalgam restorations with light-cured composites

3.3.1 Treatment of restoration

In case of composite and amalgam repair:

1. Create mechanical retention where possible. Roughen the fractured surface for best results, sandblast the area to be repaired with an intraoral microetcher (50 µm alumina). Rubber dam is recommended with high-speed evacuation.
2. Rinse with water for 10 seconds (microetched areas for 15-20 seconds), air dry.

In case of glass ceramics repair:

1. Etch with buffered hydrofluoric acid for intraoral use according to manufacturer's instructions and apply silane (e.g., Calibra® Silane Coupling Agent) to ceramic surfaces to be repaired following manufacturer's instructions.

3.3.2 Treatment of tooth

1. Treat exposed enamel and dentin surfaces according to the procedure described in chapters [3.1.1 Isolation, cleaning and conditioning of enamel and dentin](#) to [3.1.3 Acid conditioning of enamel and dentin](#).
2. Apply and cure adhesive to repair surfaces and exposed enamel and dentin surfaces described in chapters [3.1.4 Dosage](#) to [3.1.6 Light-curing of adhesive](#).
3. Proceed immediately to completion/restorative material placement following the Instructions for Use of the used composite material.

3.4 Cavity varnish for use with fresh amalgam

1. Apply and light-cure adhesive to cavity surfaces without acid conditioning as described in chapters [3.1.4 Dosage](#) to [3.1.6 Light-curing of adhesive](#).
2. Proceed immediately to completion/restorative material placement.

3.5 Cementation of indirect restorations using Calibra® Ceram or Calibra® Universal

Calibra® Ceram/Calibra® Universal.

When using Calibra® Ceram or Calibra® Universal the Dentsply Sirona Self Cure Activator is **not** required.

Light-curing.

Light-curing of Prime&Bond® universal may be accomplished right after seating the restoration with Calibra® Ceram or Calibra® Universal in case of light-transmissible indirect restorations (≤ 2.5 mm thick ceramic and composite) only.

Application of adhesive.

Application of adhesive to the inner surface of glass ceramic restorations treated with silane is not required.

1. For pulp protection and tooth conditioning, please see chapters [3.1.1 Isolation, cleaning and conditioning of enamel and dentin](#) to [3.1.3 Acid conditioning of enamel and dentin](#).
2. Apply adhesive to prepared tooth surfaces according to the procedure described in chapters [3.1.4 Dosage](#) to [3.1.6 Light-curing of adhesive](#).
3. Treat bonding surface of restoration as follows:
 - **In case of glass ceramics**, etch with hydrofluoric acid according to the manufacturer's instructions and apply silane (e.g., Calibra® Silane Coupling Agent) following manufacturer's instructions.
 - **In case of zirconium dioxide, composite or metal**, sandblast with an intraoral microetcher (50 μ m alumina). Rinse with water for 15-20 seconds, air dry. **Apply** Prime&Bond® universal adhesive to bonding surfaces of restorations to enhance adhesion. **Evaporate** as outlined in [3.1.5 Application of adhesive to enamel and dentin](#).
4. Proceed immediately to cementation of indirect restoration following the Instructions for Use for Calibra® Ceram or Calibra® Universal.

3.6 Placement of direct dual-cure/self-cure composite restorations and core build-ups

Dentsply Sirona Self Cure Activator.

Dentsply Sirona Self Cure Activator is required. The Self Cure Activator (SCA) is mixed with Prime&Bond® universal to make the latter compatible with conventional methacrylate-based dual- and self-cured composite restorative materials, including Dentsply Sirona dual- and self-cured composite restorative and cement materials.

3.6.1 Pulp protection and tooth conditioning

For pulp protection and tooth conditioning, please see chapters [3.1.1 Isolation, cleaning and conditioning of enamel and dentin](#) to [3.1.3 Acid conditioning of enamel and dentin](#).

3.6.2 Dosage and mixing

1. Dispense 1-2 drops of Prime&Bond® universal into a clean CliXdish™ Mixing Well or standard plastic mixing well, followed by Dentsply Sirona Self Cure Activator (SCA), at a ratio of 1:1 (same number of drops).
2. Mix contents for 1-2 seconds with a new Applicator Tip.

3.6.3 Application of mixed adhesive/activator to enamel and dentin

1. Use the Applicator Tip to apply mixed adhesive/activator to completely wet the surfaces to be treated. If necessary rewet Applicator Tip. Avoid pooling of the adhesive/activator.
2. Keep the mixed adhesive/activator slightly agitated for 20 seconds.
3. Disperse mixed adhesive/activator and remove solvent with clean, dry air from an air-water syringe. Treat every surface with a **moderate air flow** for at least 5 seconds until achieving a glossy and uniform layer. Avoid dry spots and splashing of mixed adhesive/activator by using a too strong air flow.

Insufficient evaporation of solvent.

Inadequate polymerization.

1. Follow strictly the instructions of the steps above.

3.6.4 Light-curing of mixed adhesive/activator

1. Light-cure adhesive/activator with a suitable curing light². Cure adhesive/activator **for 10 seconds** when using a curing light with a minimum irradiance of 800 mW/cm². Cure adhesive/activator **for at least 20 seconds** when minimum irradiance is between 550 mW/cm² and 800 mW/cm².

Insufficient curing.

Inadequate polymerization.

1. Check compatibility of curing light.
2. Check curing time.
3. Check minimum irradiance.

2. Proceed immediately to placement of the dual-cure/self-cure composite or core build-up material.

3.6.5 Placement of direct dual-cure/self-cure composite restorations and core build-ups

1. After application and light-curing of adhesive/activator, place restorative following the Instructions for Use of the used dual-cure/self-cure composite or core build-up material.
2. Ensure that the cured adhesive surfaces remain uncontaminated before applying the direct restorative material. If salivary contamination occurs, thoroughly clean with vigorous water spray, dry and reapply adhesive/activator mixture as described in chapters [3.1.4 Dosage](#) to [3.1.6 Light-curing of adhesive](#) without repeating the acid conditioning step.

² Curing light designed to cure materials containing camphorquinone (CQ) initiator. Peak of spectrum in the range of 440-480 nm and irradiance of 800-2000 mW/cm².

3.7 Cementation of indirect restorations with other dual-cure and self-cure resin cements

Dentsply Sirona Self Cure Activator.

Dentsply Sirona Self Cure Activator is required. The Self Cure Activator (SCA) is mixed with Prime&Bond® universal to make the latter compatible with conventional methacrylate-based dual- and self-cured composite restorative materials, including Dentsply Sirona dual- and self-cured composite restorative and cement materials.

Application of adhesive.

Application of adhesive to the inner surface of glass ceramic restorations treated with silane is not required.

3.7.1 Pulp protection and tooth conditioning

For pulp protection and tooth conditioning, please see chapters [3.1.1 Isolation, cleaning and conditioning of enamel and dentin](#) to [3.1.3 Acid conditioning of enamel and dentin](#).

3.7.2 Dosage and mixing

1. Dispense 1-2 drops of Prime&Bond® universal into a clean CliXdish™ Mixing Well or standard plastic mixing well, followed by Dentsply Sirona Self Cure Activator (SCA), at a ratio of 1:1 (same number of drops).
2. Mix contents for 1-2 seconds with a new Applicator Tip.

3.7.3 Application of mixed adhesive/activator to enamel and dentin

1. Use the Applicator Tip to apply mixed adhesive/activator to completely wet the surfaces to be treated. If necessary rewet Applicator Tip. Avoid pooling of the adhesive/activator.
2. Keep the mixed adhesive/activator slightly agitated for 20 seconds.
3. Disperse mixed adhesive/activator and remove solvent with clean, dry air from an air-water syringe. Treat every surface with a **moderate air flow** for at least 5 seconds until achieving a glossy and uniform layer. Avoid dry spots and splashing of mixed adhesive/activator by using a too strong air flow.

Insufficient evaporation of solvent.

Inadequate polymerization.

1. Follow strictly the instructions of the steps above.

3.7.4 Light-curing of mixed adhesive/activator

1. Light-cure adhesive/activator with a suitable curing light³. Cure adhesive/activator **for 10 seconds** when using a curing light with a minimum irradiance of 800 mW/cm². Cure adhesive/activator **for at least 20 seconds** when minimum irradiance is between 550 mW/cm² and 800 mW/cm².

³ Curing light designed to cure materials containing camphorquinone (CQ) initiator. Peak of spectrum in the range of 440-480 nm and irradiance of 800-2000 mW/cm².

Insufficient curing.

Inadequate polymerization.

1. Check compatibility of curing light.
2. Check curing time.
3. Check minimum irradiance.

2. Proceed immediately to placement of the indirect restoration.

3.7.5 Preparation of indirect restorations

Application of adhesive.

Application of adhesive to the inner surface of glass ceramic restorations treated with silane is not required.

1. Treat bonding surface of restoration as follows:
 - **In case of glass ceramics** etch with hydrofluoric acid according to the manufacturer's instructions and apply silane (e.g., Calibra® Silane Coupling Agent) following manufacturer's instructions.
 - **In case of zirconium dioxide, composite or metal**, sandblast with an intraoral microetcher (50 µm alumina). Rinse with water for 15-20 seconds, air dry. **Apply mixed adhesive/activator** to bonding surfaces of restorations to enhance adhesion. **Evaporate** as outlined in [3.1.5 Application of adhesive to enamel and dentin](#). **Cure** as outlined in [3.1.6 Light-curing of adhesive](#).
2. Proceed immediately to cementation of indirect restoration. Prepare, mix and apply dual-cure/self-cure resin cement, according to manufacturer's instructions.

3.8 Endodontic post cementation

3.8.1 With core-X® flow core build-up material

1. Post space must be rinsed with water and dried using suction and/or paper points to leave a moist surface.
Optional post space may be etch-and-rinse-treated with phosphoric acid according to the procedure described in chapters [3.1.3 Acid conditioning of enamel and dentin](#).
2. Mix Prime&Bond® universal universal adhesive with Dentsply Sirona Self Cure Activator as described in chapter [3.7.2 Dosage and mixing](#).
3. Apply mixture of Prime&Bond® universal universal adhesive and Dentsply Sirona Self Cure Activator to post space according to the procedure described in chapter [3.7.3 Application of mixed adhesive/activator to enamel and dentin](#). Use of paper points after application helps to avoid pooling of adhesive in post space.
Light-curing of adhesive before post cementation is optional – follow chapter [3.7.4 Light-curing of mixed adhesive/activator](#).
4. Treat post surface according to the post manufacturer's instructions (i.e., for X-Post® fiber post apply adhesive mixture and air-dry).
5. Proceed immediately to endodontic post cementation following the Instructions for Use for core-X® flow.

3.8.2 With other dual-cure and self-cure resin cements

1. Post space must be etch-and-rinse-treated with phosphoric acid according to the procedure described in chapters [3.1.3 Acid conditioning of enamel and dentin](#).
2. Mix Prime&Bond® universal universal adhesive with Dentsply Sirona Self Cure Activator as described in chapter [3.7.2 Dosage and mixing](#).
When using Calibra® Ceram or Calibra® Universal the Dentsply Sirona Self Cure Activator is not required.
3. Apply mixture of Prime&Bond® universal universal adhesive and Dentsply Sirona Self Cure Activator to post space according to the procedure described in chapter [3.7.3 Application of mixed adhesive/activator to enamel and dentin](#). Use of paper points after application helps to avoid pooling of adhesive in post space.
4. Light-cure adhesive before post cementation following chapter [3.7.4 Light-curing of mixed adhesive/activator](#).
5. Treat post surface according to the post manufacturer's instructions.
6. Proceed immediately to endodontic post cementation following respective Instructions for Use.

4 Hygiene

Cross-contamination.

Infection.

- Do not reuse single use products. Dispose of in accordance with local regulations.
- Reprocess reusable products as described below.

Cross-contamination.

Infection.

- The bottles cannot be reprocessed. Dispose of contaminated bottles in accordance with local regulations.
- To prevent bottles from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the bottle is handled offside the dental unit with clean/disinfected gloves. Do not reuse bottles if contaminated.
- During treatment clinicians with patient contact should not handle the bottle.

Incidental contact of the bottle with water, soap or a water-based hospital-level disinfection solution will not damage bottle body. Do not allow any solution contact with contained material. Discard material that has been in contact with any fluid or non-sterile instrument.

Repeated liquid contact may damage label. Dry the bottle with a lint-free single-use cloth.

NOTE: Vigorous wiping can destroy the label. Wipe bottles gently.

4.1 CliXdish™ Mixing Well

For reprocessing instructions, please refer to the Instructions for Use of the CliXdish™, which is available on our webpage at <https://www.dentsplysirona.com/ifu>. If requested, we will send you a free printed copy of the Instructions for Use in the language you require within 7 days. You can use the order form provided on <https://www.dentsplysirona.com/ifu> for this purpose.

4.2 Disposal

Dispose in accordance with local regulations.

5 Lot number (), expiration date () and correspondence

1. Do not use after expiration date.
ISO standard is used: "YYYY-MM" or "YYYY-MM-DD"
2. The following numbers should be quoted in all correspondence:
 - Reorder number (REF)
 - Lot number
 - Expiration date
3. Any serious incident in relation to the product should be reported to the manufacturer and the competent authority according to local regulations.
4. Device Identification (Basic UDI-DI): ++D010ADH02ME

A summary of the safety and clinical performance (SSCP) for this product can be found (upon activation) at <https://ec.europa.eu/tools/eudamed> by searching using the Basic UDI-DI number listed above and at <https://www.dentsplysirona.com/ifu> using the reference number (REF).

© Dentsply Sirona 2023-02-27

[These Instructions for Use are based on Master Version 10]

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Dentsply DeTrey GmbH